

TECHNICAL DATA SHEET



Quickfix™ Skin Glue
Topical Skin Adhesive (n-butyl-2-cyanoacrylate)

Design

- * Precision applicator with flat tip flow indicator
- * Breathable, water-resistant
- * Flexible and patient comfort
- * Fast-setting/ drying tissue adhesive
- * Vial can be crushed effortlessly to begin application
- * Smooth & controlled application
- * Comes in unique secondary PP box

HOW TO USE SKIN GLUE

1. Clean the outer layer of the wound
2. Peel open to remove the applicator
3. Keep the applicator in upright position
4. Gently press the applicator to crush the ampule
5. Gently squeeze the applicator till the flat tip shows the liquid flow
6. Apply the glue by gently squeezing the applicator
7. Hold it to 30 seconds to fix

Do not let the adhesive enter the wound while applying as it hinders wound healing.
Keep the wound horizontal in order to prevent it from running off into other areas.
Gauge bandage can be applied for protection against mechanical injury.

Technical Specifications

Chemical Name: n-butyl-2-cyanoacrylate

Glue qty: Available in 0.25 ml, 0.5 ml & 1 ml vials

Color: Dyed violet for clear visibility in application

Sterility: EO

Packaging: Box of 10 sterile units

Usage: Single-use only